



Issue Date: March 27, 2020
Revision Date: May 4, 2020

Product Use Instructions - Clean Vinyl Technology™ Protective Gown

Product: Clean Vinyl Technology™ Protective Gown incorporating an antimicrobial. Single piece, seamless, pullover style with approximately ¾-arm coverage for FDA Class 1 usage.

Use: Personal Protective Equipment (PPE). AAMI/ANSI PB70 Level 1, 2 and 3. Typical usage is patient screening, specimen collection, lab, urgent care, routine office care, lobby, security, foodservice, food manufacturing and similar settings. Not for invasive/surgical settings (FDA Class 2).

How to Use: (1) Remove from package, unfold and remove one by one. (2) Locate the perforated edge, and, while keeping your hands parallel to prevent damage, tear-off the 2.0-inch strip. This is your belt. (3) Pull over head, position comfortably and then pull belt around waist and securely tie. Your back will be partially uncovered for comfort and ergonomic considerations. (4) To remove, tear from the neckline down and outward (there is an approximate 1.0-inch slit front and back), then remove in a downward motion and step out. (5) When removed, fold outside into inside and discard appropriately.

Cleaning: Single use. If desired, hand wash with mild soap and water or mild detergent. Rinse thoroughly and air dry. Do not expose to temperatures $\geq 200^{\circ}\text{F}$. No direct contact with heat source or heating element.

Recycling and Disposal: Noncontaminated gowns can be recycled (Plastic No. 3). If exposed to bio-hazards and/or contaminated, follow the protocol established by your facility, don't recycle and keep out of the general (non-biohazard) waste stream.

Traceability: Each carton is labeled with item number, date of manufacture and lot number.

Item Nos.: NX900516/NX900518 (Frosty Clear, 50-Pack/Bulk Gaylord), NX900517/NX900519 (Translucent Yellow, 50-Pack/Bulk Gaylord), NX900520/NX900522 (Translucent Blue, 50-Pack/Bulk Gaylord).

Questions/Orders: Contact your distributor, reseller or Mississippi Polymers, Inc. directly at 800-339-9440 (Phone), 662-287-9752 (Facsimile) or customerservice@mississippipolymers.com.

**Mississippi Polymers, Inc.
2733 South Harper Road
Corinth, Mississippi 38834
Made in U.S.A.**

Clean Vinyl Technology™ trademark is owned by Len-Tex Corporation, and is used with permission under license.

No warranty, express or implied, is extended by Mississippi Polymers, Inc. and liabilities, of any kind, are disclaimed.